Monteringsvejledning (DK)

Montering af R82 fodseler

Monteringsanvisning (SV)

Montera R82 fotremmar

Monteringsveiledning (N)

Montering av R82 fotseler

Mounting instruction (GB)

How to mount the R82 Foot straps

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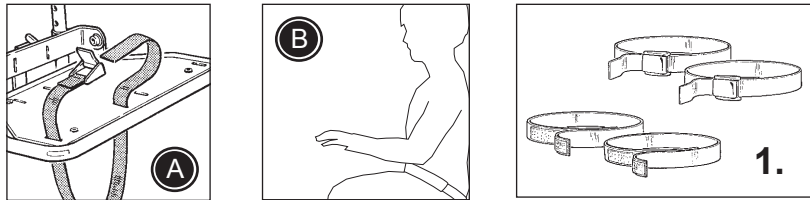
Mounting instruction

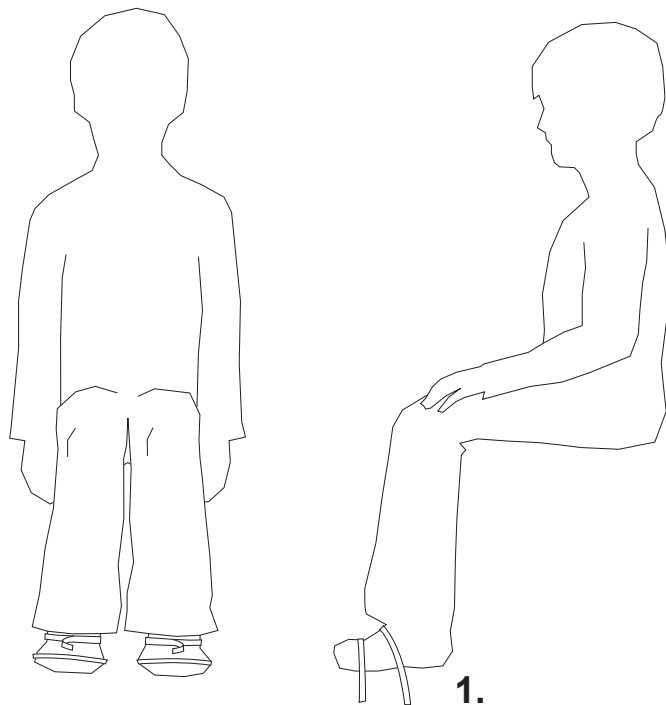
How to mount the R82 Foot straps

Recommended for: x:panda, Panda Futura, Panther, Tiger 2000, Serval, Wombat, Cheetah, Cougar, Flamingo, Swan

Mount the straps as shown on picture (A). Foot straps are available with fix locks or velcro. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The foot straps are fitted correctly when two fingers can be put between the straps and the child's foot. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (B) together with foot straps.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

 Carefully read the care label on the straps to maintain and wash the foot straps correctly.

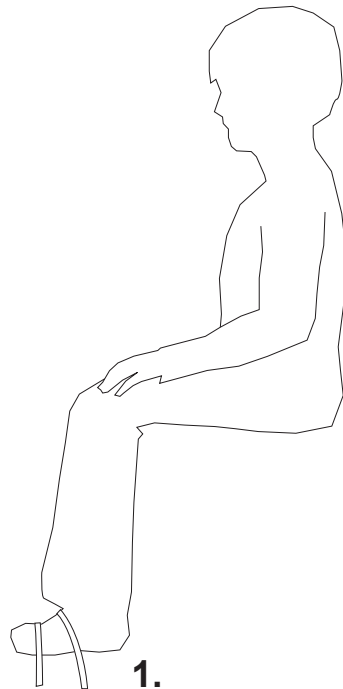
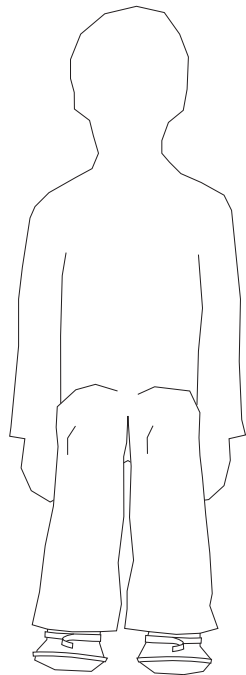
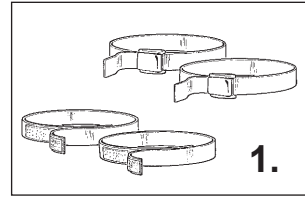
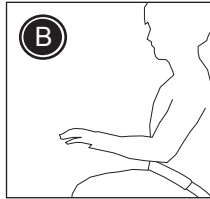
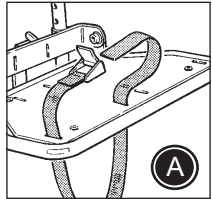
 The latest version of the mounting instruction is always available online: R82.com

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R82 A/S