Monteringsvejledning (DK)

Montering af R82 hoftesele

Monteringsanvisning (SV)

Montera R82 höftbälte

Monteringsveiledning (N)

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Mounting instruction (GB)

How to mount the R82 Hip belt

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Montage des R82 Hüftgurts

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Monteren van de R82 Heupgordel

Instruction de montage (F)

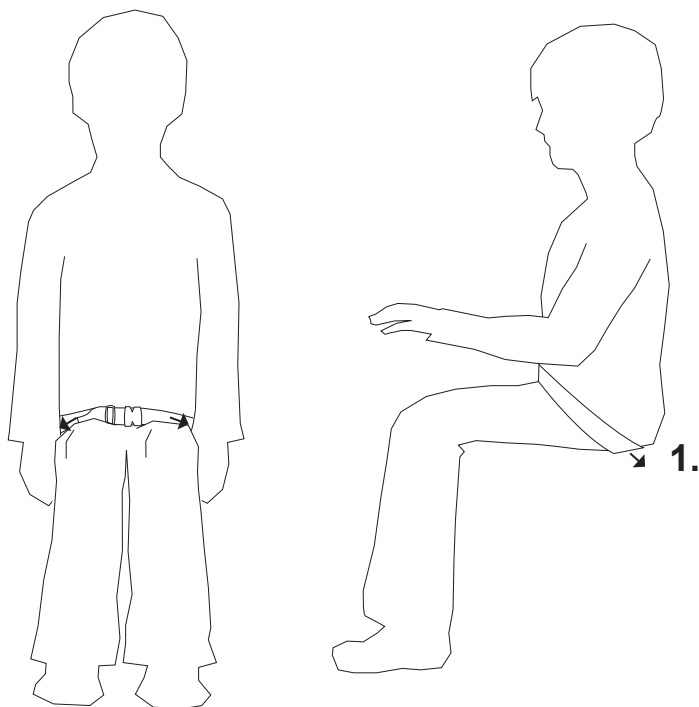
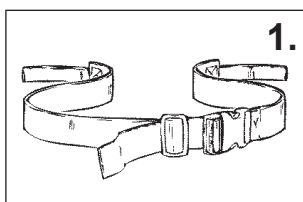
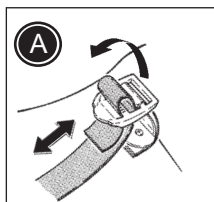
Comment monter la ceinture R82

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R82®

Montering af R82 hofsesele

Anbefalet til: x:panda, Panda Futura, Panther, Tiger 2000, Serval, Wombat, Cheetah, Cougar, Winther Reha, Flamingo, Swan

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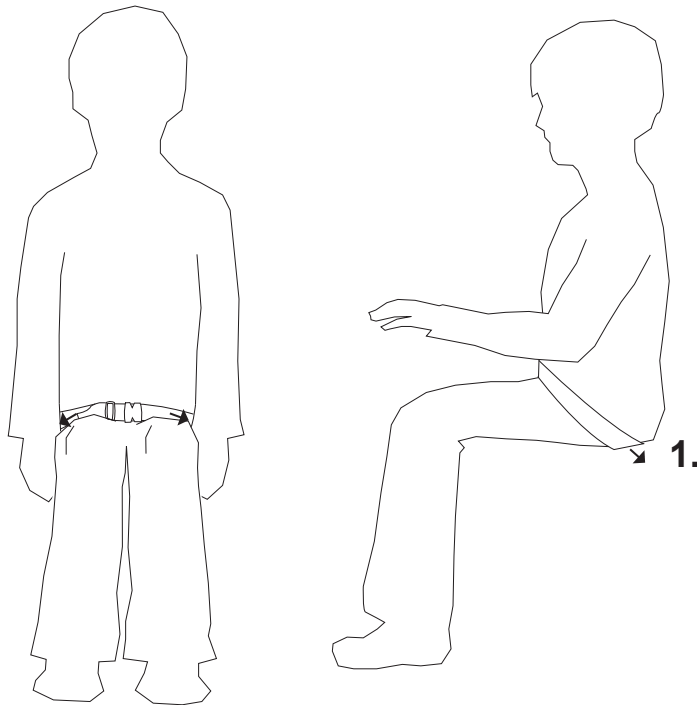
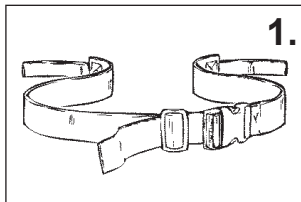
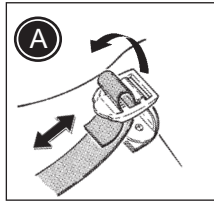
Mounting instruction

How to mount the R82 Hip belt

Recommended for: x:panda, Panda Futura, Panther, Tiger 2000, Serval, Wombat, Cheetah, Cougar, Winther Reha, Flamingo, Swan

Use the fittings (A) on the chair to mount the hip belt. If no fittings are mounted, please use the enclosed belt fittings to mount the hip belt. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

* Mount the lower straps (1) in the seat.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The hip belt is fitted correctly when two fingers can be put between the belt and the child's body. Check before every use.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

 Carefully read the care label on the harness to maintain and wash the hip belt correctly.

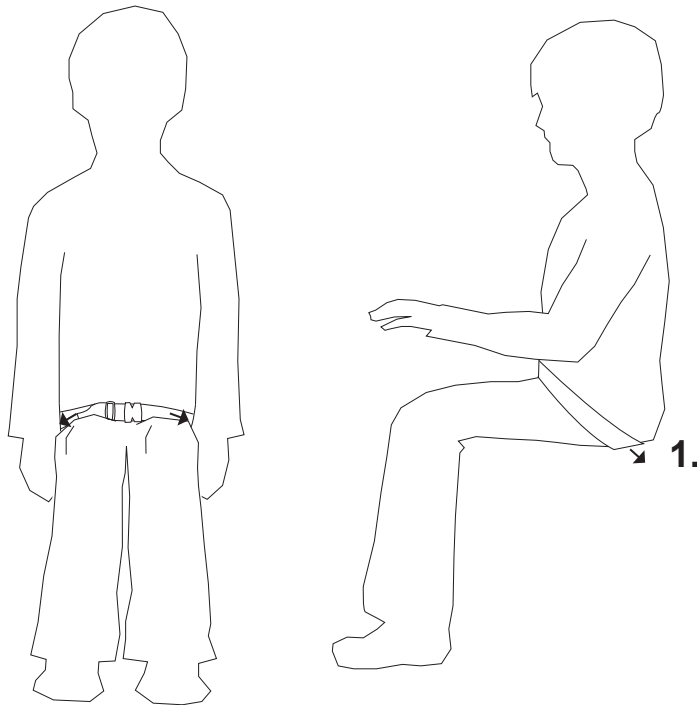
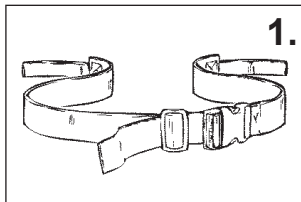
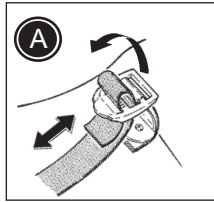
 The latest version of the mounting instruction is always available online: R82.com

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