EN We appreciate your choice of a new product from R82, read this guide before use and save it for future reference. **Intended purpose** – Vests and belts can be used in combination with medical devices and are intended to assist children and adults with disabilities to maintain the position of the body. Vest and belts can be used in combination with medical devices from R82 and other manufacturers. **Caution!** When used for: Children with limitations in their sensory function and ability to feel pain or ability to communicate pain or discomfort, where extra checks of the tension of the webbing might be needed. **Declaration of Conformity** -This product conforms to the requirements of the Medical Devices Regulation (2017/745). The CE mark must be removed if the product is modified or used for other purposes than the intended purpose. **R82 Warranty** - R82 offers a 2-year lifetime warranty against defects in workmanship and materials. The warranty would be adversely affected if the customer's responsibility of maintenance is not carried out. The warranty is only granted in the country where the product was purchased and if the product can be identified by the lot number on the label. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty is void when the product is modified or repaired. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase. **Safety** - The product should be installed and fitted by a qualified healthcare professional. In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer. 1.The product is NOT intended for use as a transportation safety device. Misuse of this product is unauthorized and unsafe. - 2.This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16. This product might loose its flame resistant characteristics if used in other products than original R82 products. - 3.It is important that the product is adjusted whenever the wheelchair or work seat is adjusted in size. 4. Always use original R82 fix-locks in combination with vest and belts. **Maintenance** - Check straps, fix-locks and stitching's for signs of wear before use and clean the product on a weekly basis. Wash the product according to instructions on washing label. Close hook and loop before washing. It is recommended to put the product in a laundry bag before washing it. In case of wear, do not alter the product. Contact an authorized R82 supplier.

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DE We appreciate your choice of a new product from R82, read this guide before use and save it for future reference. **Intended purpose** – Vests and belts can be used in combination with medical devices and are intended to assist children and adults with disabilities to maintain the position of the body. Vest and belts can be used in combination with medical devices from R82 and other manufacturers. **Caution!** When used for: Children with limitations in their sensory function and ability to feel pain or ability to communicate pain or discomfort, where extra checks of the tension of the webbing might be needed. **Declaration of Conformity** -This product conforms to the requirements of the Medical Devices Regulation (2017/745). The CE mark must be removed if the product is modified or used for other purposes than the intended purpose. **R82 Warranty** - R82 offers a 2-year lifetime warranty against defects in workmanship and materials. The warranty would be adversely affected if the customer's responsibility of maintenance is not carried out. The warranty is only granted in the country where the product was purchased and if the product can be identified by the lot number on the label. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty is void when the product is modified or repaired. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase. **Safety** - The product should be installed and fitted by a qualified healthcare professional. In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer. 1.The product is NOT intended for use as a transportation safety device. Misuse of this product is unauthorized and unsafe. - 2.This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16. This product might loose its flame resistant characteristics if used in other products than original R82 products. - 3.It is important that the product is adjusted whenever the wheelchair or work seat is adjusted in size. 4. Always use original R82 fix-locks in combination with vest and belts. **Maintenance** - Check straps, fix-locks and stitching's for signs of wear before use and clean the product on a weekly basis. Wash the product according to instructions on washing label. Close hook and loop before washing. It is recommended to put the product in a laundry bag before washing it. In case of wear, do not alter the product. Contact an authorized R82 supplier.

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© 2021 R82 A/S All rights reserved. The R82 logo and the icons are registered trademarks of R82 A/S
R82 A/S Parallelvej 3